

Vario 18



VERSATILITY AND EFFICIENCY



Vario 18 Suction Pump

Vario 18

Versatile and efficient suction

Vario 18 is an efficient medical suction pump. An all-rounder for hospitals, clinics and homecare, it offers reliability, low noise and mobility. Its innovative QuatroFlex drive unit allows for easy servicing.

Vario 18 can be used with either a Disposable or a Reusable Collection System, according to each customer's needs and demands.

Manufactured with swiss precision and quality, **Vario 18** offers a range of useful features:



- a** Overflow protection/bacteria filter
- b** Vacuum gauge in kPa and mmHg
- c** LED warning light
- d** Carrying handle
- e** Membrane vacuum regulator
- f** Tubing holder
- g** Plug for 12VDC IN car connection cable
- h** 1 L reusable collection system*

* also available with 1.5 L disposable collection system

Unique swiss technology

The innovative 4-fold piston/cylinder system, QuatroFlex, builds up the vacuum within seconds. It also convinces through compact size and low noise during operation. As well as being unique in design and technology it is easy to service and maintain. A fact, highly appreciated by medical technicians. The membrane vacuum regulator allows for precise vacuum settings. It needs to be pressed down to change the vacuum, thus making it impossible to change the settings inadvertently. The Medela Vario 18 can therefore also be used at the patient's bedside.



Precise vacuum setting with press-down knob

Versatility for a wide range of applications

Vario 18 is lightweight and portable for use across departments in hospitals and clinics and at home. It can be used for small surgical interventions as well as for airway suctioning. Variable suction levels can conveniently be set with the vacuum regulator. The Vario 18 can be equipped with either a Disposable or a Reuseable Collection System.



Optimal hygiene for many applications

Hygienic and easy use

Vario 18 offers easy, intuitive handling. All accessories are designed to complement it perfectly and to prevent false connections. Optional accessories include a trolley, carrying bag and various attachments. It also features an inbuilt battery charger that allows for mobile use. The need to recharge is made audible and visible by special battery alarms.



Can be equipped for individual needs

Fast Facts

- Medela offers healthcare professionals 50 years of expertise in medical vacuum technology.
- Our swiss development and manufacturing process yields mechanical precision to guarantee excellent performance.
- QuatroFlex technology: silent function: can be used in environments in which quiet operation is necessary.
- Precise vacuum settings and fast vacuum build-up: ready when needed.
- One device for multiple needs.
- New users need little instruction to operate.
- Mains-independent for 30 minutes.
- Acoustic and optical battery management.
- Wide range of accessories available for individual application.
- Vario 18 incorporates reliability and ease of use like the Medela suction pumps for operating theatre.
- 2 years warranty on device.

Delivering excellence for over 50 years with
comprehensive, high quality Swiss medical vacuum systems.

Accessories

Filters

to protect the pump from overflow, the environment from bacteria and to neutralise odour.



Trolley

with rails for mobile transport of the pump and accessories.



Reusable Collection System

autoclavable jars and lids collect secretions easily and economically. Sizes available: 1 and 2 litres.



Holders and plates

can be used to attach Vario 18 to standard rails.



Disposable Collection System

with disposable suction liners and reusable PC jars that offer a hygienic, simple handling and efficient fluid management. Sizes available: 1.5 and 2.5 litres.



Carrying bag

for transporting the pump.



Use Medela accessories for even greater flexibility and ease of use.

Plug for 12 VDC IN car connection cable

for operating the pump in the car.



Technical data



High vacuum
-75kPa
-563 mmHg



AC 3,5 kg
AC/DC 4,2 kg



IP21



low flow
18 l/min.



H x W x D
380 x 170 x 285 mm



ISO 9001
ISO 13485
CE (93/42/EEC), IIa
2 years warranty



CAUTION: Federal US law restricts this device to sale by or on the order of a physician.

Medical Vacuum Technology for Healthcare Professionals

Please contact us or your local Medela representative for details.



Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

Local contact:

Australia
Medela Pty Ltd.
Medical Technology
3 Arco Lane
Heatherton, Vic, 3202
Australia
Phone +61 3 9552 8600
Fax +61 3 9552 8699
contact@medela.com.au
www.medela.com.au

Canada
Medela Inc.
4160 Sladeview Cres., Unit #8
Mississauga, ON
L5L 0A1
Canada
Phone +01 905 608 7272
Fax +01 905 608 8720
info@medela.ca
www.medela.ca

India
Medela India private limited
c/o Vatika Business Park
First floor, tower 2,
Sohna Road, Sec-49,
Gurgaon 122 002
India
Phone +91 124 4416999
Fax +91 124 4416990
info@medela.in
www.medela.in

UK
Medela UK Ltd.
Huntsman Drive
Northbank Industrial Park
Irlam, Manchester M44 5EG
UK
Phone +44 161 776 0400
Fax +44 161 776 0444
info@medela.co.uk
www.medela.co.uk

USA
Medela Inc.
P.O. Box 660
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
info-healthcare@medela.com
www.medela-healthcare.us