

Disinfection wipes

DESCOCEPT SPEZIAL WIPES

Alcohol-free wipes for rapid disinfection

- Universal, comfortable and timesaving application in a usual dispenser bucket
- Basic, rapid efficacy
- Especially suited for disinfection of large surfaces in routine areas



Product description

Alcohol-free, ready-to-use disinfection wipes for rapid disinfection and cleaning of medical inventory and all kinds of surfaces in medical areas as well as in all areas of the food industry and in canteens. Based on quaternary ammonium compounds DESCOCEPT SPEZIAL WIPES are free from alcohol and do not contain aldehydes or phenols. Due to their high-quality lint-free material composition, guaranteed release of active ingredients and excellent cleaning power, DESCOCEPT SPEZIAL WIPES are especially suitable for all professional applications.

Range of application

According Biocidal Regulation (BPR):

For rapid disinfection and cleaning of alcohol sensitive medical equipment and surfaces of every type.

According Medical Device Directive (MDD):

For rapid disinfection and cleaning of alcohol sensitive, non invasive medical devices.

Application

Wipe surfaces with DESCOCEPT SPEZIAL WIPES until completely wet and allow to dry for the entire contact time.

Spectrum of efficacy and contact times			30 s	1 min	2 min
Bacteria and yeasts					
application recommendation for surface disinfection: bactericidal, yeasticidal*	VAH** / EN 16615	with mechanical action, dirty conditions		•	
Enveloped viruses					
active against enveloped viruses	RKI/DVV	incl HBV/HIV/HCV	•		
	EN 14476	clean conditions	•		
Non-enveloped viruses					
active against papova/SV40 viruses	EN 14476	dirty conditions			•
active against rotaviruses	RKI/DVV		•		

*including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)
**listed in the 5 min column of the VAH list

EN standards			1 min
Antimicrobial properties			
bactericidal	EN 1040	basis test	•
yeasticidal (C. albicans)	EN 1275	basis test	•
bactericidal (medical area)	EN 13727	dirty conditions	•
yeasticidal (C. albicans, medical area)	EN 13624	dirty conditions	•
bactericidal (food and industrial area)	EN 1276 / EN 13697	dirty conditions	•
yeasticidal (C. albicans, food and industrial area)	EN 1650 / EN 13697	dirty conditions	•



DESCOSEPT SPEZIAL WIPES

Alcohol-free wipes for rapid disinfection

Application notes

Shelf life after opening: 28 days.

Composition

100 g solution contain:

0.5 g Alkyl(C12-16)dimethylbenzyl ammonium chloride,

0.25 g didecyldimethylammoniumchloride.

Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.

Precautionary and hazard statements

Harmful to aquatic life with long lasting effects. Avoid release to the environment. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Single unit	Content	Delivery unit	Size	REF
DESCOSEPT SPEZIAL WIPES	Wet Wipe Dispenser	100 wipes	1	32 x 30 cm	00-208-ERS100
	Refill pack	100 wipes	3	32 x 30 cm	00-208-RS100

CE 0482



Use disinfectants safely. Always read label and product information before use.

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH. www.schumacher-online.com